

Topi-CLICK[®] MICRO[™]

User Instructions



Targeted Topical
Micro-dosing


- ✓ Automated-metered topical dosing provides an accurate and consistent dose every time
- ✓ Audible, Tactile and Visual CLICKs indicate measure of dose (no need to manually mark dose)
- ✓ Innovative TRIAD shape allows for precision handling
- ✓ Built-in REFILL Badge helps patients reorder medication on-time



Easy
Screw
Cap

STEP 1:

Unscrew the cap of the Topi-CLICK[®] Micro[™] and set aside for recapping after use.

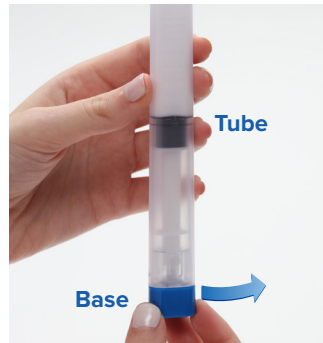

Tube
Base

STEP 2:

Holding the Micro[™] in an upright position, turn/CLICK the **Base** the number of CLICKs or mLs specified on the prescription label. Use the Audible, Tactile and Visual CLICK points of the triad shaped **Base** as dosing indicators. Be sure to align **Base** with the **Triad Tube** to ensure dose is fully dispensed.



STEP 3:

Apply topical medication with built-in applicator as directed by physician or pharmacist. **Continue rubbing until preparation is completely absorbed into the skin.** **IMPORTANT:** The built-in applicator lessens the risk of transference to unintended areas. No need to use your fingertips to apply medication.



Screw-on

STEP 4:

Screw-on cap after each dose to prevent evaporation. Be sure cap is secured completely.



REFILL Badge[™]

REFILL
Line

Plunger

The Refill Badge[™] reminder system helps patients reorder in a timely manner.

When the **TOP** black line of the **Plunger** reaches the embossed line below the word "REFILL" approximately 32 CLICKs are left. Approximately 12 CLICKs remain at the line above the word "REFILL".

Note: Follow instructions on prescription label before use. Topi-CLICK[®] Micro[™] is not re-fillable.

Made of FDA grade materials | Phthalate free | UV block and is Recyclable ♻️.