

## Topi-CLICK<sup>®</sup> MICRO<sup>™</sup>

## User Instructions




Targeted Transdermal  
MICRO-Dosing

- ✓ Automated-metered transdermal dosing provides an accurate and consistent dose every time
- ✓ Audible & Tactile and Visual CLICKs indicate measure of dose (no need to manually mark dose)
- ✓ Innovative TRIAD shape allows for precision handling
- ✓ Built-in REFILL Badge helps patients reorder medication on-time



**STEP 1:**  
Unscrew the cap of the Topi-CLICK<sup>®</sup> MICRO<sup>™</sup> and set aside for recapping after use.

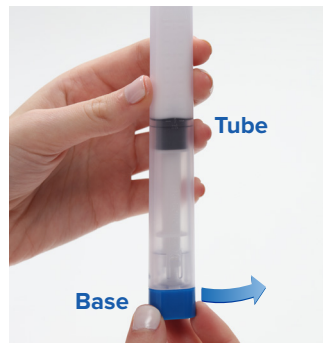

**STEP 2:**  
Holding the MICRO<sup>™</sup> in an upright position, turn/CLICK the **Base** the number of CLICKs or mLs specified on the prescription label. Use the Audible, Tactile and Visual CLICK points of the triad shaped **Base** as dosing indicators. Be sure to align **Base** with the **Triad Tube**.



**STEP 3:**  
Apply transdermal medication with built-in applicator as directed by physician or pharmacist. **Continue rubbing until preparation is completely absorbed into the skin.**  
**IMPORTANT:** The built-in applicator lessens the risk of transference to unintended areas. No need to use your fingertips to apply



**STEP 4:**  
Screw-on cap after each dose to prevent evaporation. Be sure cap is secured completely.



REFILL  
Line

Plunger

### The Refill Badge<sup>™</sup> reminder system helps patients reorder in a timely manner.

When the **TOP** black line of the **Plunger** reaches the embossed line below the word "REFILL" approximately 32 CLICKs are left. Approximately 12 CLICKs remain at the line above the word "REFILL".

*Note: Follow instructions on prescription label before use. Topi-CLICK<sup>®</sup> MICRO<sup>™</sup> is not re-fillable.*

Made of FDA grade materials | Phthalate free | UV block and is Recyclable ♻️.