

# Topi-CLICK<sup>®</sup>






Accurate & easy dosing  
for pets



CLICK... Lick...  
goi<sup>™</sup>



Choose Topi-CLICK<sup>®</sup> for The Best User Experience<sup>™</sup>

-  Automated metered-dosing
-  Built-in dosing applicator
-  Audible, Tactile and Visual CLICKs indicate measure of dose (no need to manually mark dose)
-  Pets learn to associate the CLICK as positive conditioning so there is no struggle when dosing
-  Built-in REFILL Badge helps pet owners reorder medication on-time

## Lickable Suggestions



Amlodipine  
Atenolol  
Ciprofloxacin  
Doxycycline  
Fluconazole  
Fluoxetine  
Gabapentin  
Methimazole  
Mirtazepam  
Metronidazole  
Phenobarbital  
Piroxicam  
Potassium Bromide  
Ursodiol

Follow instructions on provider's label before using the Topi-CLICK<sup>®</sup>. Consult with literature or a prescriber veterinarian on the most appropriate route of delivery and dosing before dispensing to patients.



Request FREE Topi-CLICK<sup>®</sup> 35 samples for patient trials and also get a copy of the accuracy study at <http://DoseLogix.com/samples>.


CLICK... Lick...  
go!

# TOPI-CLICK®

Accurate & easy dosing for pets



## Topi-CLICK® Pet Use Instructions

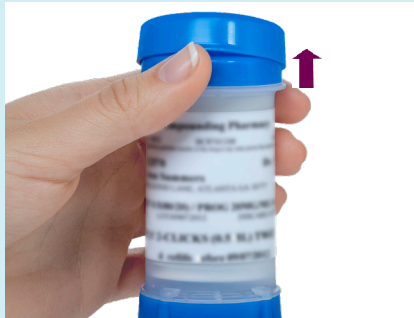

### STEP 1:

Lift off the cap of the Topi-CLICK® container by placing your thumb under the tab on the protective outer cap and pressing upward.



### STEP 2:

Turn the base to dispense the number of CLICKS specified in the instructions on your prescription label.



### STEP 3:


Watch the visual indicators when rotating base. Be sure to leave the base in the registered CLICK position by aligning the base column with the tube node after each use.

**Each turn/CLICK of the base dispenses 0.25 mL of medication accurately.**



### STEP 4:

Hold Topi-CLICK® while your pet licks the applicator pad to get an accurate tasty dose of medication.

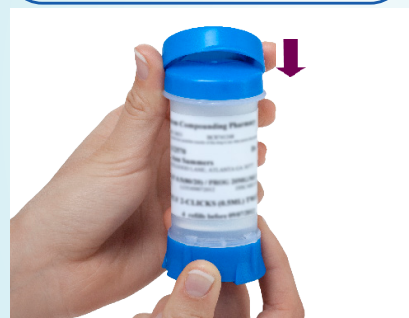

### STEP 5:

Replace cap after each dose to prevent evaporation. Be sure cap snaps on tightly.

**Note** Be sure to align ports when using the Topi-CLICK® 35 3Port™ cap and applicator.



### STEP 6:

The bottom of the plunger is the indicating line; when it reaches the embossed line below the word "Refill" approximately 32 CLICKS are left. Approximately 12 CLICKS remain at the line above the word "Refill".

*Follow instructions on provider's label before using the Topi-CLICK®. Consult with literature or a prescriber veterinarian on the most appropriate route of delivery and dosing before dispensing to patients.*

**<sup>1</sup>Topi-CLICK® 35 is the most accurate topical dosing dispenser among those compared in an independent study titled *Topical Metered Dosing Dispenser Performance Evaluation* by Analytical Research Laboratories. Request study at [http://doselogix.com/accuracy\\_study/](http://doselogix.com/accuracy_study/).**

Topi-CLICK35VetUse\_021120