


# Topi-CLICK® 35 Quick Advancing Plunger Instructions for Smaller Fill Volumes

When less volume space is needed for Topi-CLICK® filling (i.e. filling 18ml instead of 35ml), there is an easy way to advance the Topi-CLICK® plunger.

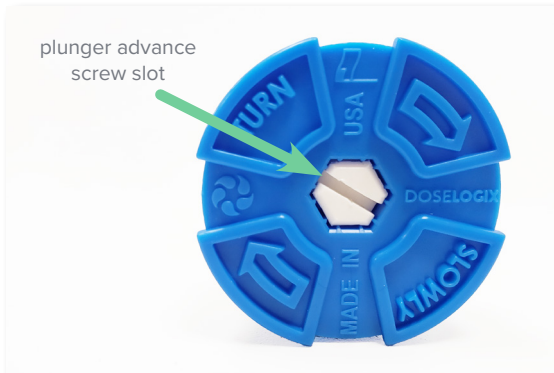

## STEP 1: Advance Screw Slot

On the bottom of the Topi-CLICK® dispenser, locate the plunger advance screw slot. The screw slot on the bottom of the Topi-CLICK® will fit a standard flat-head screwdriver.\*

\*We recommend using a mini cordless screwdriver without a reversible trigger (200 RPM max).



## STEP 2: Turning the Screw

Insert the screwdriver into the plunger advance screw slot and turn it clockwise (following the direction of the arrows). Staying in control, turn the screwdriver until the plunger is advanced to the desired height.

Warning: Turning the screw too fast or in the counter-clockwise direction can damage the unit. High-speed electric drills are not recommended.



## STEP 3: Proceed to Filling

The plunger is now set to the desired height for smaller volumes. Your Topi-CLICK® 35 is now ready for filling!

Read all information, specifications, and instructions carefully before filling and dispensing Topi-CLICKs®.

